

DESCOCEPT AF LEMON

alcoholic
rapid disinfectant

- ✓ with fresh lemon scent
- ✓ free from aldehydes
- ✓ free from phenols

- aldehyde-free rapid disinfectant with lemon scent
- comprehensive virucidal efficacy incl. Noro virus
- fragrance variant of Descosept AF



applicable
as foam



CE 0482



PRODUCT PROPERTIES

DESCOCEPT AF LEMON is a ready-for-use, aldehyde-free rapid disinfection based on ethanol and quaternary ammonium compounds.

DESCOCEPT AF LEMON has a pleasant fresh scent and is well tolerated by the skin. Discolourations of materials after the application of **DESCOCEPT AF LEMON** are virtually impossible due to its aldehyde-free composition.

DESCOCEPT AF LEMON is comprehensively effective against viruses incl. Noro viruses.

Suitable as a surface disinfectant in all areas where a rapid disinfection is required, e.g.:

- dental and medical practices (e.g. disinfection of treatment chairs)
- for ophthalmologists and opticians to prevent viral infections
- rapid disinfection of equipment in the operating theatre, incl. disinfection of the operating table between operations
- bedsteads, mattresses, blankets and pillows
- ambulances and garbage cans
- general inventory like trays, trolleys and baskets for transport of e.g. bandaging material, bath-tubs, examination couches, toilet seats
- clothes like overalls, shoes used in the operating theatre, rubber aprons and general laundry

RANGE OF APPLICATION

Acc. to MDD (Medical Device Directive): Rapid disinfection of alcohol resistant non invasive medical devices.

Acc. to Biocidal Products Regulation: Rapid disinfection and cleaning of alcohol resistant medical inventory as well as of all kinds of surfaces. Especially suitable for inaccessible surfaces.

APPLICATION

DESCOCEPT AF LEMON can be applied by wiping or spraying. Surface disinfection by wiping (recommended procedure): Apply a sufficient quantity of **DESCOCEPT AF LEMON** to a suitable single-use non-woven wipe and wipe surfaces. Surfaces must remain completely wet for the required contact time. Surface disinfection by spraying (recommended only for difficult to reach surfaces): Spray **DESCOCEPT AF LEMON** onto surfaces from a distance of at least 30 cm. Surfaces must remain completely wet for the required contact time. After contact time has elapsed, surfaces may be wiped dry with a clean single-use wipe if required.

DESCOCEPT AF LEMON can be used with the **DESCO WIPES DT** dispenser system for a convenient and user-friendly application with a proven disinfectant efficacy and standing time of up to 28 days.

ANTIMICROBIAL PROPERTIES

DESCOCEPT AF LEMON is

- bactericidal (incl. MRSA)
- tuberculocidal
- yeasticidal
- virus inactivating incl. HBV/HIV, HCV, BVDV, Rota-, Adeno-, Vaccinia viruses, Noro viruses

CONCENTRATIONS & EXPOSURE TIMES

bactericidal incl. MRSA, yeasticidal

acc. to DGHM/ÖGHMP/VAH 1 min*
(*listed in the 5-minutes-column of the DGHM/VAH)

TB bacteria (*M. terrae*) 1 min
Aspergillus niger 30 min


active against all enveloped viruses acc. to RKI-recommendations such as Vaccinia virus, BVDV, HBV, HIV, HCV, Influenza viruses 30 sec
Rota-/Adeno viruses 1 min
Noro virus (MNV) 5 min



DR. SCHUMACHER GMBH
Am Roggenfeld 3, D-34323 Malsfeld
Telephone +49 5664 9496-0

WWW.SCHUMACHER-ONLINE.COM

DESCOCEPT AF LEMON alcoholic rapid disinfectant

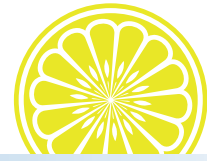

CONTACT TIMES ACC. TO EN

soiling	EN 1040 Bacteri- cidal	EN 1275 Yeastici- dal	EN 1276 Bacteri- cidal	EN 1650 Yeastici- dal	EN 13697 Bacteri- dal	EN 13697 Yeastici- dal	EN 14476 Noro Virus
low	1 min.	30 sec.	1 min.	1 min.	1 min.	1 min.	5 min

COMPOSITION

100 g solution contain:

42.0 g ethanol

0.05 g didecyldimethyl ammonium chloride

Contains perfumes.

CHEMICAL-/PHYSICAL PROPERTIES

Appearance:	clear, colourless solution
pH-value:	7.0 +/- 1.0
Flash-point:	25 °C

DELIVERY UNITS

single unit	delivery unit	PZN	REF-No.
1 L bottle	10 x 1 L	1899267	00-311L-010
5 L container	3 x 5 L	1899296	00-311L-050
10 L container	1 x 10 L	1899391	00-311L-100

APPLICATION AIDS

Foamspray head for 1-litre-bottle, spray head for 500 ml and 1- litre-bottle, transfer tap for container, container key.

SPECIAL INFORMATION

Flammable. Keep container tightly closed - No smoking. Keep away from sources of ignition! In case of contact with eyes rinse immediately with plenty of water and seek medical advice. Do not breathe vapour/aerosol. Use only in well ventilated areas. Application max. 50 ml per m². The total amount of application per room must not be exceed 100 ml per m² of floor space. Keep out of reach of children.

Use disinfectants safely. Always read label and product information before use.

EXPERT OPINIONS

Dr. Pitten, Hygienist, Gießen:

Expert's report on surface disinfection acc. to DGHM/VAH-guidelines.

Dr. Suchomel, Hygienist, Vienna:

Expert's report on surface disinfection acc. to DGHM/VAH-guidelines.

Dr. Steinmann, Virologist, Bremen:

Expert's report on adeno virucidal activity; vaccinia virucidal activity; hepatitis C virucidal activity (BVDV).

Prof. Dr. Werner, Hygienist, Schwerin:

Expert's report on the tuberculocidal activity.

Expert's report on Noro virucidal (MNV) activity acc. to EN 14476.

Expert's report on bactericidal and yeasticidal efficacy acc. to EN 1040, EN 1275, EN 1276, EN 1650 and EN 13697.

Expert's report on disinfection efficacy/ standing time in combination with **DESCO WIPES DT** and **ECO WIPES** dispenser systems.

Dr. Brill, Hygienist, Hamburg:

Examination of the microbiological stability of **DESCOCEPT AF** in combination with **DESCO WIPES DT**; Disinfection activity of the non-woven material **DESCO WIPES DT** in combination with **DESCOCEPT AF**. Microbiological stability of **DESCOCEPT AF** with **ECO WIPES**.

Dr. Voss, Dermatest, Münster:

Expert's report on skin tolerance (epicutane test).

Konrad Hornschuch AG, Weißbach:

Manufacturer's approval for use with Skai® cushion materials.

Bergé & Co. GmbH, Versmold:

Manufacturer's approval for use **DESCOCEPT AF** with MM disinfection mats.

PRODUCT STATUS/LISTINGS

Listed in the current disinfectant list of the VAH/DGHM and ÖGHMP.

Listed in the IHO list of virucidal products (www.iho-viruzidie-liste.de).

Conforms to the EU-guideline 93/42/EC for medical devices.

Registered as a biocide product at BAuA (German Institute for Employment Protection and Occupational Medicine):

BAuA reg. No.: N - 55153.

ENVIRONMENTAL INFORMATION

The products of Dr. Schumacher GmbH are manufactured according to modern, safe and environmentally friendly processes. Due to the compliance with high quality standards the constant and very good product quality can be guaranteed. The company is certified according to DIN EN ISO 13485, DIN EN 9001, DIN EN 14001 & BS OHSAS 18001 and has a validated eco management system (EMAS).



DR. SCHUMACHER GMBH
Am Roggenfeld 3, D-34323 Malsfeld
Telephone +49 5664 9496-0

WWW.SCHUMACHER-ONLINE.COM